

1. True or False, you should have the donor wash their hands prior to the collection? (Circle)
2. The Rapid Drug Screen results should be read:
 - a. Immediately
 - b. After 10 minutes
 - c. Between 5-8 minutes
 - d. None of the above
3. True or False, two lines = a positive result? (Circle)
4. The absence of any lines in the results window means:
 - a. Invalid test, recollect another specimen
 - b. Negative result
 - c. Positive result
 - d. The specimen was adulterated
5. A negative result is recorded when:
 - a. No lines appear
 - b. Two lines appear
 - c. One line appears
 - d. None of the above
6. The Rapid Drug Screen has a shelf life of
 - a. One month
 - b. Six months
 - c. 1 Year
 - d. 18 months
7. True of False, all positive screens should be confirmed? (Circle)
8. The temperature strip should be read?
 - a. Between 5-8 minutes
 - b. After 10 minutes
 - c. Before running the test
 - d. Immediately upon receipt of collection cup from the donor
 - e. C & D
9. If the temperature strip is not within the acceptable range, you should?
 - a. Ask the donor to provide a new specimen
 - b. Consider this an invalid specimen
 - c. A & B
 - d. None of the above
10. True or False, THC is the chemical present in Marijuana? (Circle)
11. Examples of Opiates include?
 - a. Heroin
 - b. Codeine
 - c. Poppy seeds
 - d. All of the above

12. True or False, someone who takes over-the-counter sinus medication could show up positive for Amphetamines? (Circle)
13. True or False, the collection cup should be as full as possible? (Circle)
14. True or False, the test card should be pushed down until touching the bottom of the cup? (Circle)
15. The urine level in the cup should be?
- To the top of the cup
 - Half full
 - No higher than the blue line
 - None of the above
16. The expiration date and lot number of each kit is located?
- On the bottom of the cup
 - On the top of the foil packet
 - There is no expiration date
 - There is no lot number
17. If the donor fills the collection cup to the top, you should?
- Run the test immediately
 - Not run the test at all
 - Have the donor provide a new specimen
 - Have the donor pour out until urine reaches the blue area
18. If the initial screen is positive, you should?
- Scream out I found one
 - Accuse the donor of being a drug addict
 - Run a second Rapid Drug Screen to be sure
 - Follow your agency's policy for confirmation testing
19. True or False, some over-the-counter medications may cause a positive result?(Circle)
20. True or False, only prescription medications will cause a positive? (Circle)
21. True or False, only illegal drugs will cause a positive test result? (Circle)

FAX COMPLETED EXAM TO 813-661-4754